# Tinearity™ G1

User Manual - UK

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Doc-10363 rev 2

Date of issue: 2022-08-23 / Ausstellungsdatum: 23.08.2022

Utfärdet den: 2022-08-23 / Udstedelsesdato: 2022-08-23

**CE 2862**

User Manual - UK .....	5-23
Användarhandbok - SE .....	27-45
Bedienungsanleitung - DE .....	49-68
Brugervejledning - DK.....	71-89

## The User Manual

The Tinearity device is intended for relief and for treatment for people suffering from Tinnitus. The device is intended to be used in homes, at work as well as in public environments. Make sure that warnings and cautions related to specific situations mentioned in the "Important information" section are considered.

This user manual gives guidance on how to use and maintain your new Tinearity device.

Please read through this manual carefully to optimize the use of Tinearity Sound Generator. Pay extra attention to the **Warnings section**, the **cautions** and **Maintenance**. If any questions, contact your country representative.

## Warning symbols used in this manual

 **WARNING** Indicates a potentially hazardous situation which, if not avoided, could result in injury.

 **CAUTION** Indicates a potentially hazardous situation which, if not avoided, could result in damage to environment or properties.

 **Note** Indicating useful information about the safe use of the device.

## Contents

1. Introduction .....	7
2. Intended use .....	7
3. Indications for use .....	7
3.1. Intended medical indication .....	7
3.2. Contraindications .....	8
3.3. Possible side effects .....	8
3.4. Useful life / service life .....	8
3.5. Suitability of the treatment .....	8
4. System overview .....	8
4.1. Packaging content .....	8
5. Tinearity G1 description .....	9
6. Device description .....	9
6.1. Sound generator .....	9
6.2. Adapter .....	10
6.3. Charger .....	10
6.4. Packaging material .....	10
7. Preparation before first use .....	11
8. Preparation before every use .....	11
8.1. Skin hygiene .....	11

8.2.	Sound generator and Adapter.....	12
8.3.	Charger.....	12
<b>9.</b>	<b>Operating instructions user .....</b>	<b>12</b>
9.1.	Applying the adapter.....	12
9.2.	Connecting the Sound generator .....	14
9.3.	Turn the Sound generator on and off.....	14
9.4.	Adjust the volume.....	14
9.5.	Treatment.....	14
9.6.	Remove the Sound generator .....	14
9.7.	Remove the adapter.....	15
<b>10.</b>	<b>Maintenance .....</b>	<b>15</b>
10.1.	Preparation for cleaning .....	15
10.2.	Cleaning the Sound generator .....	15
10.3.	Cleaning the Charger.....	16
<b>11.</b>	<b>Charging the batteries / sound generator .....</b>	<b>16</b>
11.1.	General information .....	16
11.2.	When to re-charge batteries/sound generator.....	16
11.3.	How to charge the sound generator.....	16
<b>12.</b>	<b>Consumables, Service and Repair.....</b>	<b>17</b>
<b>13.</b>	<b>Technical description .....</b>	<b>17</b>
13.1.	Trouble shooting .....	17
13.2.	Technical data.....	17
<b>Pollution degree .....</b>		<b>17</b>
<b>14.</b>	<b>EMC information.....</b>	<b>18</b>
<b>15.</b>	<b>Manufacturer's warranty.....</b>	<b>20</b>
<b>16.</b>	<b>Important information .....</b>	<b>20</b>
16.1.	Warnings.....	20
16.2.	Caution.....	21
16.3.	Contacts.....	21
<b>17.</b>	<b>Description of symbols used in this user manual.....</b>	<b>22</b>

## 1. Introduction

Model REF: 6000, Tinearity™ G1

### Name and address to the manufacturer

Duearity AB  
Krusegränd 42 D  
212 25 Malmö  
Sweden

### Name and address to the country/region representatives

Duearity AB  
Krusegränd 42 D  
212 25 Malmö  
Sweden

## 2. Intended use

Tinearity G1 is a device that generate white noise to relieve and treat patients suffering from tinnitus. The target population is adult population over 18 years of age.


 **Note** The patient is the intended operator of the device.

## 3. Indications for use

### 3.1. Intended medical indication

Tinearity G1 is intended to treat and relieve Tinnitus in users with normal hearing or mild hearing loss. Tinearity G1 have a performance output as shown in diagram 1. Users is recommended to get help from an audiologist to establish their audiogram.

*Diagram 1. Maximum output performance*

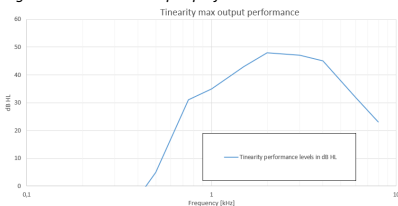The recommended treatment time is 8 hours per day for 6-24 months.

### 3.2. Contraindications

Users with moderate and severe hearing loss is contraindicated, i.e., hearing threshold level above 40dB HL.

Users suffering from Hyperacusis is contraindicated.

### 3.3. Possible side effects

The adapter is made up of a plastic holder and a medical grade tape. The tape has an acrylic adhesive that attaches the adapter to the user's skin. An area of irritation might appear behind the ear from switching the single use adapter.

### 3.4. Useful life / service life

Useful life of the sound generator and the charger is 24 months, calculated from the manufacturing date. Expiration date for the adapter is stated at the label.

### 3.5. Suitability of the treatment

The treatment can require involvement of healthcare professional, if the healthcare professional require information regarding the suitability of the treatment, further information can be obtained by contacting the manufacturer.

---

## 4. System overview

Your Tinearity G1 device is designed to relief and treat your symptoms associated with tinnitus by the use of white noise. The white noise is by the sound generator converted into vibrations that are transmitted via the adapter through the skull to the inner ear. This method works independently of the ear canal and the middle ear and facilitates the treatment to be performed without obstructing your normal hearing canal. Thus, your Tinearity G1 device can be used at all times during the day, during work, during activities as well as during sleep.

 **CAUTION** The sound generator shall never be cleaned or submerged in or under water or other liquids.

### 4.1. Packaging content

- Tinearity G1 Sound Generator, one pair (2pcs)
- Tinearity G1 Adapters (62pcs)
- Tinearity G1 Charger
- USB cable

 **Note** The Power supply for connection of the charger to the wall socket is not included in the packaging, see section technical data.

 **WARNING** The Tinearity G1 device include small parts that may be an asphyxiation risk for small children. Always keep the charger station, the sound generator and the adapter out of reach for children.


 **WARNING** The USB cable is included in the packaging and might pose a risk of choking, keep away from small children.

## 5. Tinearity G1 description

Figure 1. Device overview.

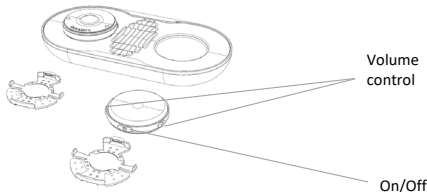Tinearity™ G1 comprises three components: (1) the sound generator, (2) adapters and (3) a charger (including a USB cable for connection to a power supply). The sound generator is attached to the skin behind the ear by means of the adapter.

The sound generator works within the frequency span of 1kHz-10kHz with a maximum output level of 48dB HL. This device is intended for use during eight (8) hours a day when set at the maximum output level. The sound generator is supplied in pairs, with the intention to be used at both sides of the head. The adapter is a disposable device that serves as a mechanical connector between the sound generator and the user. The adapter is made up of a plastic holder that are compatible with the sound generator and a medical grade tape to be attached to the user's skin. The adapter is a single use device and is designed to be removed daily after each treatment.

The sound generator uses a re-chargeable battery as a power source which is charged with the charger. The charger consists of a charger plate designed for two sound generators to be charged simultaneously and a separate USB cable with a USB-C port for connection to the wall socket.

The Tinearity G1 sound generator and adapter can be used at all time during the day, during sleep as well as during work and spare time.

**⚠ CAUTION** The sound generator shall never be cleaned or submerged in or under water or other liquids.

## 6. Device description

### 6.1. Sound generator

The sound generator convert white noise into vibrations. It is driven by a re-chargeable li-ion battery.

**⚠ CAUTION** The sound generator contain electrical parts and shall be disposed as electrical waste.

**👉 Note** for explanation of the symbols see section 17 "Description of symbols used in this user manual".

Figure 2. Sound generator

Figure 5. Primary package label Sound Generator.

Figure 6. Primary package label Adapter.

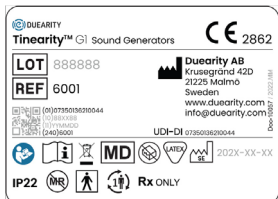
Figure 7. Customer label Tinearity System.

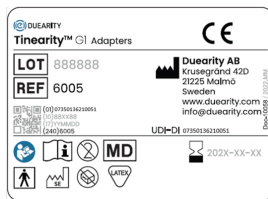
Figure 8. Customer package label Adapters

**Note** for explanation of the symbols see section 17 “Description of symbols used in this user manual”.

## 7. Preparation before first use

- Unpack the parts, check visually for any damages and follow the cleaning instructions for the sound generator and charger according to section 10.2 and 10.3.
- Check safety-related labeling on the Tinearity G1 devices for legibility.
- Make sure that your power supply (not supplied in the package) for connecting the charger into the wall socket is the following performances:
  - USB-A port, 5V DC, min 1000mA, Max 1500mA
  - Double insulated
  - CE marked
- Charge the sound generators completely, using the supplied charger, see section 11.

## 8. Preparation before every use

### 8.1. Skin hygiene

The skin behind the ear must be clean, dry and free from hair. If the hairless area is not large enough, adjust the hairline with a razor.

Clean the skin with mild soap and let it dry completely before you apply the adapter.

Figure 9.

## 8.2. Sound generator and Adapter

Check the sound generator and the adapter visually before use, for any damage.

**⚠ WARNING** If covers are damaged or any parts are loose, do not use the device.

Clean the sound generator according to section 11.

## 8.3. Charger

Place the device on a flat surface.

**⚠ WARNING** Check the cable and connections for pinching or damage. Pets and pests may damage the insulation of the cables. Do not use damaged cables.

---

# 9. Operating instructions user

## 9.1. Applying the adapter

- 1) Locate the area for application of the adapter behind the ear. To maximize the performance of your treatment make sure that the adapter is placed at the mastoid bone and orientated as illustrated in below figure. Make sure that the area is free from hair.

Figure 10. Note the adapter has a single curvature designed to be follow the curvature behind the ear, therefore correct rotation on the application of adapter is advised to ensure a good adhesion.

Figure 11. Correct application, the soft part is applied closest to the ear.

Figure 12. In-correct application, rotate the adapter 180 degrees relative your skull.

Figure 13. In-correct application, rotate the adapter 90 degrees relative your skull.

➡ **Note** Transfer of sound/noise through Bone conduction is most effective if the adapter is attached to the skin where the tissue thickness is minimized.

- 2) Hold the adapter in the wings.
- 3) Remove the protective film, do not touch the adhesive surface.

Figure 14.

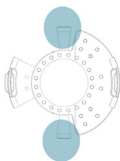
Figure 15.

- 4) Press gently on the highlighted surface of the adapter for at least 20 seconds.

Figure 16.

⚠ **WARNING** Risk for skin damage. Only use the adapter provided by Duearity and according to this manual.

## 9.2. Connecting the Sound generator

To safely and comfortably connect the sound generator to the adapter, tilt it slightly and carefully push it onto the adapter until it clicks into place.

 **Note** The side of the sound generator marked with CE<sub>2862</sub> shall be towards the adapter.

Figure 17.

Figure 18.

## 9.3. Turn the Sound generator on and off

Turn your sound generator on and off by pressing the “power” button. A short press will turn your sound generator on and a longer press will turn your sound generator off.

Figure 19.


 **Note** use a short press to turn your device on, a long press will turn your device off.

## 9.4. Adjust the volume

Adjust the volume until you feel a relief from your tinnitus noise.

- Turn the volume up by pressing the button marked with “+”
- Turn the volume down by pressing the button marked with “-”

Figure 20.

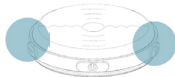## 9.5. Treatment

Use your sound generator for 8h each day. The total treatment period shall not exceed 24 months.

 **WARNING** Risk of hearing damage. If you **perceive discomfort** or if your tinnitus is **getting worse**, immediately **abort** the treatment and consult a healthcare professional.

## 9.6. Remove the Sound generator

Remove the Sound generator by holding the sound generator and firmly pressing the wing handle furthest away from the ear on the adapter. The sound generator will now disconnect.

Figure 21.

**⚠ CAUTION** Risk of damage to the device, charge the transducer after each use.

### 9.7. Remove the adapter

Remove the adapter by slowly peeling it off from the skin. The adapter shall be removed after each use. It is for single-use only.

Figure 22.

**⚠ WARNING** Risk of skin damage, only use the adapter provided by Duearity and according to this manual.

**👉 Note** Dispose the adapter after each use as plastic waste.

---

## 10. Maintenance

Store your Tinearity G1 device in room temperature protected from dust, dirt and sunlight when you do not use the device. Before each use, follow these cleaning instructions.

**👉 Note** Your sound generator is protected against ingress of water but is not waterproof.

### 10.1. Preparation for cleaning

- Separate your sound generator from the adapter.
- Secure that your sound generator is turned off.
- Remove the USB cable from the charger

**👉 Note** the adapter is single use and shall be disposed after each use/treatment, no cleaning is needed.

### 10.2. Cleaning the Sound generator

Clean your sound generator on the outside with soft detergent wipes, as minimum every 7 days, the duration of the cleaning action shall be minimum 1 minute. Make sure to reach all sides, gaps and ridges. The level of cleaning shall be as minimum equivalent to general personal grooming items.

**⚠ CAUTION** Your sound generator is protected against ingress of water, i.e., it is designed to be used in your daily life but the sound generator is not waterproof.

**⚠ CAUTION** The sound generator shall never be cleaned or submerged in or under water or other liquids.

### 10.3. Cleaning the Charger

Clean your charger on the outside with soft detergent wipes, as minimum every 7 days, the duration of the cleaning action shall be minimum 1 minute. Make sure to reach all sides, gaps and ridges. The level of cleaning shall be as minimum equivalent to general personal grooming items.

**⚠ CAUTION** The charger shall never be cleaned in or under water or other liquids.

---

## 11. Charging the batteries / sound generator

### 11.1. General information

The sound generator is driven by a Li-Ion battery. The sound generator is rechargeable and have an operating period of maximum 12 hours, depending on the used amplitude level.

### 11.2. When to re-charge batteries/sound generator

Your sound generator shall be re-charged after each use.


**⚠ WARNING** The sound generator include coin cell batteries that may be corrosive if swallowed. Risk for small children. Always keep the sound generator out of reach for children. If a battery is accidentally swallowed, seek medical attention at the nearest emergency department.

### 11.3. How to charge the sound generator

To charge the sound generator, follow below steps:

- Make sure that the Charger is placed in a room that has a temperature of maximum 35 C
- Place your sound generator(s) with the front side (indicated with the DUEARITY symbol) facing the inductive charger, see figure 23.
- The blinking green light indicates that the device is charging.
- Let the sound generator(s) charge until the indicator(s) stops blinking. A complete charge cycle is approximately 6 hours. If the battery is fully discharged or if the ambient temperature exceeds 30 degC during charging, the charging will be prolonged, but not entirely stopped in order to prevent damaging the battery or overheating, respectively.
- Your device(s) is/are now fully charged and ready for use.

Figure 23.

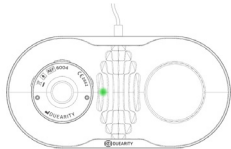**👉 Note,** Risk of damage to the device, charge the sound generator after each use.

## 12. Consumables, Service and Repair

If your Tinearity device is not functioning properly, please visit your MyPage and follow the instructions. If you need to order an additional set of adapters, please visit your MyPage and follow the instructions.

## 13. Technical description

### 13.1. Trouble shooting

Table 1.

Symptom	Explanation	Possible Cause	Proposed action
No white noise is perceived.	The amplitude of the white noise must exceed the hearing threshold of the user.	Hearing impaired.	Increase the amplitude (turn up the volume), see section 9.4.
No white noise is perceived.	The batteries must be charged after each use.	The batteries are discharged.	Charge your sound generator, see section 11.
The sound generator falls off from the adapter.	The adapter is worn.	The adapter is single use.	Change adapter, see section 9.1 and 9.7.

### 13.2. Technical data

Table 2.

Environmental conditions	Sound generator	Inductive charger
<b>Ambient air pressure</b>		
Operation	700 hPa to 1060 hPa	
Transport/ storage		
<b>Ambient temperature</b>		
Operation	+5 to +35 C	
Transport and storage	-25 to +70 C	
<b>Ambient humidity</b>		
Operation	RH 15% to 93% , non condensing	
Transport and storage		
Operational altitude	Max 3000 m over see level	
<b>Classification</b>		
Class	II	III
Applied part category	BF	Non
Protection of harmful ingress	IP 22	IP 21
Pollution degree	II	II
Mode of operation	Continuous use	Continuous use
Isolation from supply mains	N/A	Disconnect AD / DC adapter
Nominal voltage	3,7 VDC	5 VDC

Environmental conditions	Sound generator	Inductive charger
Battery capacity	50 mAh	No battery
Battery type	Rechargeable Li-ion battery.	No battery

#### Requirement for AC/ DC adapter

Table 3.

Output power voltage	5VDC
Output power	Min. 1000mA, Max 1500mA
IP class	Minimum IP 21
Class	II
Approval	Certified to IEC 62368

## 14. EMC information

Table 4.

Electromagnetic Emissions	
<p>The Tinearity system is intended for use in the electromagnetic environment associated with a Professional healthcare facility or a Home healthcare environment as specified below. The customer or the user of the Tinearity system should assure that they are used in such environment.</p>	
Emission Tests	Compliance
Radiated and conducted RF emissions EN55011/CISPR 11	Sound Generator: Class B, Group 1 Charger: Class B, Group 2  <b>Electromagnetic environment - guidance</b> The Tinearity system uses RF energy only for their internal function. Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment.
Harmonic emissions IEC 61000-3-2	<b>Sound generator:</b> Not applicable, battery operated device <b>Charger:</b> Not applicable, P < 75W
Voltage fluctuations / flicker emissions IEC 61000-3-3	<b>Sound generator:</b> Not applicable, battery operated device <b>Charger:</b> Not applicable, P < 75W

Table 5.

Electromagnetic Immunity			
<p>The Tinearity system is intended for use in the electromagnetic environment specified below. The customer or the user of Tinearity system should assure that they are used in such environment.</p>			
Immunity Tests	Basic EMC standard or test method	Immunity test levels	
		Professional healthcare facility environment	Home healthcare environment
Electrostatic Discharge	61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	
Radiated RF EM fields	61000-4-3	3 V/m 80 MHz - 2.7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz - 2.7 GHz 80 % AM at 1 kHz

Proximity fields from RF wireless communications equipment	61000-4-3	Test frequency (MHz)	Service	Immunity Test Level (V/m)
		385	TETRA 400	27
450	GMRS460, FRS460	28		
710 745 780	LTE Band 13, 17	9		
810 870 930	GSM 800/900, TETRA800, iDEN 820, CDMA 850, LTE Band 5	28		
1720 1845 1970	GSM 1800, CDMA 1900, GSM 1900, DECT, LTE Band 1, 3, 4, 25, UMTS	28		
2450	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	28		
5240 5500 5785	WLAN 801.11 a/n	9		
Surges	IEC 61000-4-5	<b>Sound generator:</b> Not applicable, battery operated device <b>Charger:</b> Tested through generic AC/DC adapter Line-to-line: $\pm 0.5$ kV, $\pm 1$ kV		
Conducted disturbances	IEC 61000-4-6	<b>Sound generator:</b> Not applicable, battery operated device <b>Charger:</b> Tested through generic AC/DC adapter		
		3 V/m 0,15 MHz – 80 MHz 6 V/m in ISM bands  80 % AM at 1 kHz	3 V/m 0,15 MHz – 80 MHz 6 V/m in ISM and amateur radio bands 80 % AM at 1 kHz	
Rated power frequency magnetic fields	61000-4-8	30 A/m 50 Hz		
Voltage dips	IEC 61000-4-11	<b>Sound generator:</b> Not applicable, battery operated device <b>Charger:</b> Tested through generic AC/DC adapter 0% UT, 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0% UT, 1 cycle at 0° 70% UT, 25 cycles at 0°		
Voltage interruptions	IEC 61000-4-11	<b>Sound generator:</b> Not applicable, battery operated device		

## 15. Manufacturer's warranty

Duearity sound generators and inductive chargers are covered by a limited warranty issued by the manufacturer. The warranty period is 24 months from the date of delivery. This limited warranty covers manufacturing and material defects in the sound generator or charger itself.

Problems arising from improper handling or care, excessive use, accidents, repairs or exposure to corrosive conditions, are NOT covered by the limited warranty and may void the warranty.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing the sale of consumer goods.

## 16. Important information

### 16.1. Warnings

-  **WARNING** Risk of hearing damage. If you **perceive discomfort** or if your tinnitus is **getting worse**, immediately **abort** the treatment and consult a healthcare professional.
-  **WARNING** Risk of electrocution. Your Sound processor has achieved IP 22 classification and is referred as water-resistant, not water-proof. Always remove your Sound processor before showering or bathing.
-  **WARNING** Risk of electrocution. Use cables, chargers, adapters and sound generators provided by the manufacturer, **ONLY**.
-  **WARNING** Risk of body damage. Never expose your Tinerity G1 device for extreme heat, do not dry the device in microwave or other ovens.
-  **WARNING** Risk of electrocution or burns,
  - Do not open the device
  - Do not use device if the enclosure or cable is damaged
  - Do not service or repair the device
  - No modification of this equipment is allowed
  - Only use power supply that fulfills the specification in this instruction
-  **WARNING** Risk of infection. This device is to be used by one user and shall not be reused by several individuals
-  **WARNING** Risk of fire and hazardous smoke. Do not use in an Oxygen rich environment or in areas with flammable gases.
-  **WARNING** Risk of affecting measurement results and treatment results. This device emits electromagnetic radiation, do not use closer than 30 cm from other medical devices.
-  **WARNING** Risk of hearing damage. Patient suffering from Hyperacusis (sound over sensitive) or other underlying illnesses shall consult physician before treatment is started.
-  **WARNING** Risk of burn injury. Do not use during MRI (Magnetic Resonance Imaging) examination.
-  **WARNING** Risk for damage to your skin (irritation or redness from the use of the adapter). It is important to follow the skin hygiene guidelines. Please contact your local general practitioner if you have any concerns. Only use the adapter provided by Duearity and according to this manual.

- ⚠ WARNING** Risk for hearing damage. The Device is not to be used by persons under the age of 18 years old.
- ⚠ WARNING** The Tinearity G1 device include small parts that may be an asphyxiation risk for small children. Always keep the charger, the sound generator and the adapter out of reach for children.
- ⚠ WARNING** Check the cable and connections for pinching or damage. Pets and pests may damage the insulation of the cables. Do not use damaged cables.
- ⚠ WARNING** The sound generator include coin cell batteries that may be corrosive if swallowed. Risk for small children. Always keep the sound generator out of reach for children. If a battery is accidentally swallowed, seek medical attention at the nearest emergency department.
- ⚠ WARNING** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- ⚠ WARNING** Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- ⚠ WARNING** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the [ME EQUIPMENT or ME SYSTEM], including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- ⚠ WARNING** Risk for hearing damage. If the Tinearity G1 device is combined with a hearing aid, the combination needs to be evaluated by a health care professional.

## 16.2. Caution

- ⚠ CAUTION** The power source in your sound generator has insufficient energy to cause fire under normal conditions of use. The sound generator has not been tested for compliance with international standards concerning explosive atmospheres. It is recommended not to use your sound processor in areas with danger of explosion.
- ⚠ CAUTION** No part of the device shall be cleaned in or under water or other liquids.
- ⚠ CAUTION** The sound generator and the charging station contain electrical parts and shall be disposed as electrical waste.

## 16.3. Contacts

For assistance, if needed, in setting up, using or maintaining the Tinearity system; or to report unexpected operation or events, please find contact information, depending on your location, at [www.Duearity.com](http://www.Duearity.com)

If any serious incident has occurred in relation to the device it shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 17. Description of symbols used in this user manual.

Table 6.

Symbol	Title	Description
	CE mark	The device complies with all required EU regulations and directives. The four digit number indicates the identification of the notified body
	Manufacturer	The device is produced by the manufacturer whose name and address are stated next to the symbol.
	Use-by-date	Indicate the date after which the device is not to be used.
	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalogue number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Serial number	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Country of manufacturer	Identifies the country of manufacture of products.
	Do not use if package is damage and consult instructions for use	Indicates that the device should not be used if the package has been damaged or opened and that the user should consult the instructions for use (user manual) for additional information.
	Fragile, handle with care	Indicates a device that can be broken or damaged if not handled carefully.
	Keep dry	Indicates a medical device that needs to be protected from moisture.
	Temperature limitation	Indicates the temperature limits to which the medical device can be safely exposed.
	Humidity limitation	Indicates the Humidity limits to which the medical device can be safely exposed.

	Atmospheric pressure limitation	Indicates the Atmospheric pressure limits to which the medical device can be safely exposed.
	Do not re-use / Single use	Indicates a device that is intended for one single use only.
	Single patient use	Indicates a device that is intended for one patient only.
	Consult instructions for use or consult electronic instructions for use	Consult instructions for use for warnings and cautions.
	IFU	Consult instructions for use for warnings and cautions.
	Medical Device	The device is a medical device.
	Electronic waste (Weee)	Waste from electronic equipment must be handled according to local regulations.
	Type BF Applied Part	Classification of protection against electrical shock.
IP 21	IP 21	Indicates the class of protections against harmful ingress of water and objects. IP21 indicates protected from touch by fingers and objects greater than 12 millimeters and vertically falling drops.
IP 22	IP 22	Indicates the class of protections against harmful ingress of water. IP 22 protects from dripping water, from vertical direction.
	MR Unsafe	The device is not safe in MR environment, i.e., magnetic resonance imaging (magnet camera, MR-camera), and shall be removed.
Rx ONLY	Prescription device	The device is sold with prescription. Applicable for the US market.

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